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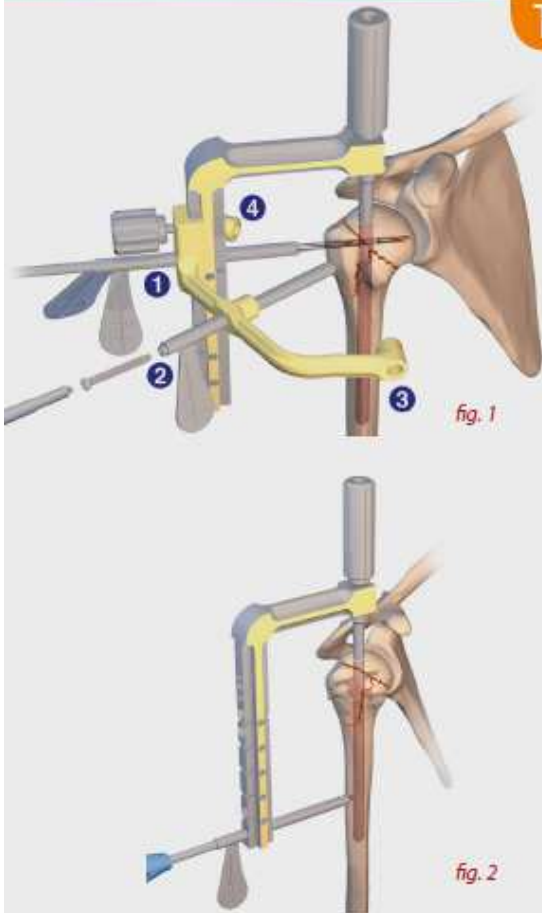
TELEGRAPH® IV

WARNING:

In case of screwing difficulty with very strong bone, **DON'T FORCE**, remove the screw and proceed to screw tapping on the first cortical bone.

Proximal locking (fig. 1)

- **Step n°1:** Place the large and small drill guides (*drill bushing - ref. 253 677 and screw tap bushing - ref. 253 678*) use the drill bit to prepare the screw **1**. **DON'T REMOVE THE DRILL BIT**, it'll stabilize the nail position.
- **Step n°2:** Use the drill bit to prepare the next screw, remove the drilling sleeve and place the screw. The graduated drill bit indicates the length (generally 40mm). The screw is inserted proximally without threading using the blue handle small fragment screwdriver (*ref. 233 339*).
- **Step n°3:** Proceed to other screws preparation and insertion if needed. Remove the first drill bit and the drilling sleeve to place the screw **1**.

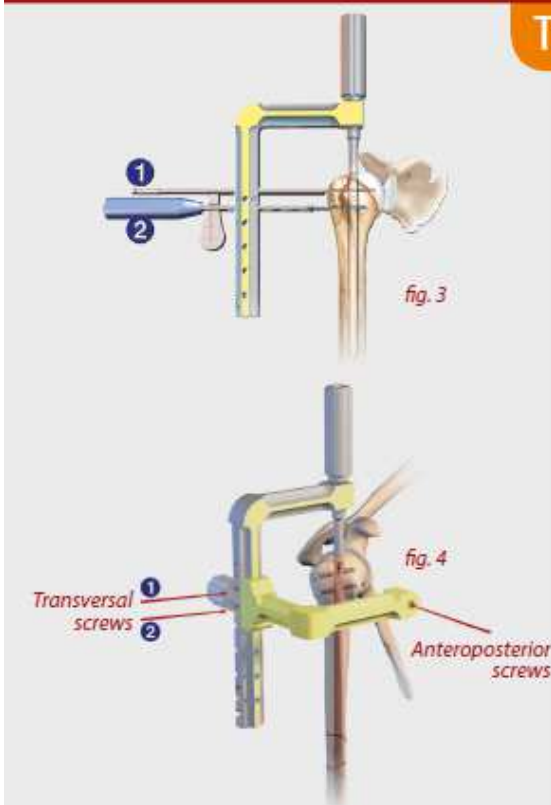
Distal locking (fig. 2)

This is recommended to be carried out dynamically (the most distal opening from the main beam).

The soft tissue retractor (*obturator - ref. 236 844*), inserted into the large drill guide, percutaneously making contact with the diaphyseal cortical bone. The next surgical stages are:

- boring 2.5mm (*ref. 250 855*);
- threading (*ref. 250 858*), mandatory here (cortical bone);
- screwing.

TELEGRAPH® SHORT / MIXTE

Nail installation and proximal locking

The nail, mounted on the nail holder (*fig. 3*), is inserted. Passage of the fracture site is generally easy. Height adjustment and proximal locking are standard.

How to proceed, proximal locking:

- Use the drill bit to prepare the screw **1**. **DON'T REMOVE THE DRILL BIT**, it'll stabilize the nail position.
- Use the drill bit to prepare the screw **2**, remove the drilling sleeve and place the screw (*fig. 4*).
- Place the screw **1** (*fig. 4*).
- Proceed to place the sagittal screw if needed (*fig. 4*).

The 2 transversal screws are placed using the ancillary device of the main beam (*fig. 4*). The anteroposterior screw is implanted using a specific arm (recommended).

N.B.

Be careful to position the nail according to the marking at the top:
- **F** for **FRONT** (anterior)
- **B** for **BACK** (posterior)

TECHNICAL SHEET - LOCKING

LONG TELEGRAPH®

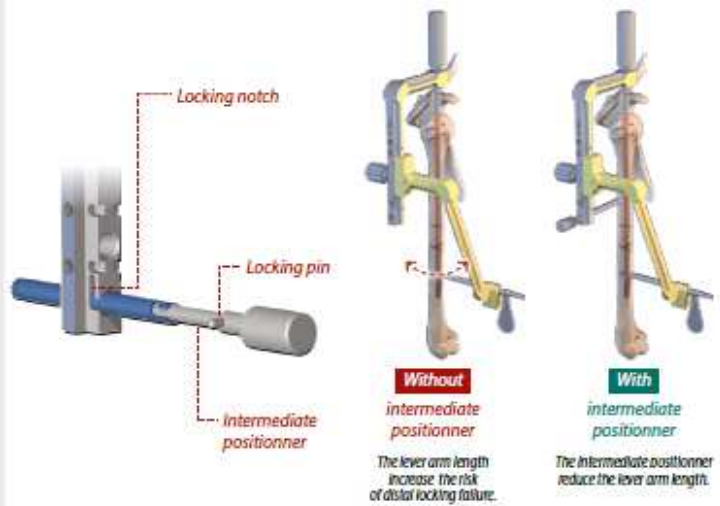
Positioning of the patient

The patient is in a semi-seated position. The affected shoulder should extend clearly beyond the edge of the operating table. The forearm should be resting on an arm-rest. The arm is in 25-30° retropulsion in relation to the thorax to liberate the point of entry. The fluoroscope should be placed longitudinally at the level of the patient's head.

Intermediate positionner to reduce lever arm

Use of intermediate positionner: an axis stress applied on the long nail can create a lever arm and thus compromise the distal locking. To reduce this lever arm, it is recommended to use the TELEGRAPH intermediate positionner.

Drilling using the oblique view

Control the drilling using the Image Intensifier.

Tips & tricks: proximal screws insertion

In order to guarantee the fitting **on a dense bone**, it is recommended to tap the first cortical bone to facilitate the screw introduction. It is important not to apply stress to the nail holder during screwing to ensure the screw fixation in the nail.

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