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EN octenilin® Wound Irrigation Solution

Instructions for use

1. Introductory remarks: Chronic skin wounds are almost always covered with exudate, a thick sequestered fibrin layer, necrotic tissue and cellular debris. This covering provides good conditions for the growth not only of pathogens, but also of transient or resident pathogens of the skin flora, considerably delaying wound healing. Careful wound cleansing and wetting of the wound with octenilin® Wound Irrigation Solution is therefore indicated.

Due to its low surface tension, octenilin® Wound Irrigation Solution has excellent wetting properties and thus cleansing properties on tissue surfaces, even difficult to reach surfaces, such as fissures, and wound pockets.

2. Product profile and applications: octenilin® Wound Irrigation Solution can be used, e.g.

- for cleansing and moistening of chronic skin wounds
- for tissue-sparing wetting of bandages and wound coverings such as compresses, gauze, cushions, sponges, gels, hydrofibres, alginates, hydrocolloids etc.
- when changing bandages, to free encrusted bandaging material or other encrusted wound coverings

The use of octenilin® Wound Irrigation Solution causes no pain and is also well tolerated by persons with allergies. Unpleasant odours are quickly removed

3. Application in general: *Only for external use on skin wounds.* octenilin® wound irrigation solution is suitable for repeated and long term use. The long term use of octenilin® wound irrigation solution should be done under medical supervision. Wounds should always be irrigated and cleansed with octenilin® Wound Irrigation Solution to remove easily soluble encrustations before further treatment with octenilin® Wound Gel (see separate instructions for use).

Bandages or dressings soaked with octenilin® Wound Irrigation Solution can be used for cleansing.

Bandages are frequently encrusted and stick to wound surfaces. If these bandages are removed when dry, it may cause new damage in the wound area, further delaying healing. In cases where bandages are difficult to remove, it is advisable to wet or soak them with octenilin® Wound Irrigation Solution until it is possible to gently remove them without traumatising the wound surface.

Application should be repeated frequently until all encrustations or necrotic tissue are easily removed and the wound looks clean. This is a good precondition for natural wound healing and ensures optimal visualisation for surgical debridement.

Patients with chronic wounds are often temperature sensitive. octenilin® Irrigation Solution can be warmed to body temperature *immediately before use it.*

4. Cleaning around the wound: Thorough extensive cleaning of the entire area surrounding the wound with octenilin® Wound Irrigation Solution is recommended to minimise the risk of transmitting pathogens to the wound.

5. Tissue tolerability and biocompatibility: non-irritating, non-sensitising, pain-free, without tissue toxicity, no impairment of granulation and epithelialisation. The good tolerability of Octenidine has been confirmed by clinical experience over many years and by clinical studies also on chronic wounds.

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7. Interactions: Do not use in combination with anionic surfactants or other wound cleansing soaps, ointments, oils, enzymes etc. since this may adversely affect the preservative. Do not use in combination with PVP-iodine, since this may cause discoloration and impair the antiseptic effect of the PVP-iodine.

8. Contraindications: octenilin® Wound Irrigation Solution should not be used in patients with known or suspected allergy to one or more of the ingredients. If in doubt consult a physician.

octenilin® Wound Irrigation Solution should not be used on hyaline cartilage, the eyes, ears, nose, urinary bladder and in the abdominal cavity! Do not use for infusion or injection! Must not be taken by mouth.

To prevent possible tissue injury, the product must not be injected or applied to tissues with pressure. Adequate drainage from wound cavities must be provided (e.g. for flexible drain tube).

9. General safety instructions: Only to be applied to the wound externally. Only flawless and undamaged container must be used.

Use only intact and undamaged packs. Protect the product against exposure to direct sunlight. To prevent the introduction of bacteria when using octenilin® Wound Irrigation Solution, ensure that the packaging does not come into contact with the wound.

- Medical device - Keep out of the reach and sight of children!

10. Composition/ technical data: Aqua valde purificata, Glycerin, Ethylhexylglycerin, Octenidine HCl. Appearance and odour: clear, colourless and nearly odourless aqueous solution.

Stability at room temperature, see stability information.

Original condition: **[STERILE]** – sterile, sealed.

Duration of use: 8 weeks after the bottle has been opened
Disposal: Fully emptied packs can be disposed of as household waste.

Conservation and duration of use: octenilin® Wound Irrigation Solution is preserved in such a way that the risk of spreading pathogens to surrounding tissue is markedly reduced when it is used.

octenilin® Wound Irrigation Solution, when used as instructed, thus constitutes an effective barrier between the wound and the treated person. Although sterility is compromised once the bottle has been opened due to the manner of conservation an 8-week period of use is possible as long as the bottle is closed immediately after each time it is used and the bottle opening is protected against contamination or direct contact. Bottles that have been in contact with the wound must be discarded.

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EN octenilin® Wound Gel

Instructions for use

Introductory remarks: chronic skin wounds are almost always covered with exudate, a thick desquamated fibrin layer, necrotic tissue and cellular debris. These coverings provide good conditions for the growth not only of pathogens, but also of transient or resident pathogens of the skin flora, which may considerably delay natural wound healing. Careful wound cleansing and a moist wound environment is therefore required.

Product profile and indications: octenilin® Wound Gel can be used, for example: • for moistening and cleansing of chronic skin wounds • for preserving moisture of dressings and wound coverings • for loosening caked coverings • for helping the natural healing process

The application of octenilin® Wound Gel is painless and is also well tolerated by persons with allergies.

3. General usage: *Only for external use on skin wounds.* octenilin® Wound Gel is suitable for repeated and long term use. The long term use of octenilin® Wound Gel should be done under medical supervision.

Accompanying the debridement, the wound should always first be irrigated and cleaned so that easily detachable slough is already removed before continuing treatment of the wound with octenilin® Wound Gel.

Cover the wound completely with octenilin® Wound Gel (1-5 mm). To cover or fill the wound, octenilin® Wound Gel can be applied to poultices, gauzes, compresses and other absorbent materials or wound filling materials.

Moistening the wound surface allows slough to be gently loosened and removed at the next dressing change.

octenilin® Wound Gel can remain on the wound until the next change of dressing. The frequency of dressing change depends on the nature of the wound and the amount of exudate and should be adapted to the current wound situation, but should be done after max. 3 days. *If used with paraffin gauzes, the dressing is to be changed and the wound cleaned daily.* The wound dressing should be chosen to suit the phase of wound healing. The secondary dressing should be chosen to ensure the gel does not dry out but so that the wound margins and the skin around the wound do not become macerated.

Continue using until all the slough and necrotic debris can be easily removed and the wound is cleansed.

Patients with chronic wounds are often sensitive to temperature. octenilin® Wound Gel can be warmed to body temperature *immediately before use.*

Cleaning the periwound area: To minimise the risk of introducing bacteria into the wound, thorough, extensive cleansing of the entire wound area with octenilin® Wound Irrigation Solution is recommended.

Tissue compatibility and biocompatibility: Non-irritating; non-sensitising; pain-free in use, no tissue toxicity with inhibition of granulation and epithelisation. The good tissue tolerability of octenidine dihydrochloride has been demonstrated during many years of use and by clinical trials with octenisept® including on chronic wounds.

Side effects: no side effects have been observed so far.

Interactions: When used with with paraffin gauzes, the gauze may dry out and become difficult to remove. To prevent the paraffin gauze drying out, a dressing change with wound cleansing should be performed daily. The dressing can be saturated with octenilin® Wound Irrigation Solution to facilitate removal.

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