Addendum

Infant Warmer System Heater Reflector Screws Maintenance Check

For future reference, add this addendum to your
Infant Warmer System Service Manual.

2098189-001 A

© 2016 General Electric Company

Language Disclaimer

(EN)

WARNING:

This service document is available in English only.

- If a customer's service provider requires a language other than English, it is the customer's responsibility to provide translation services.
 - Do not attempt to service the equipment unless this service manual has been consulted and is understood.
 - Failure to heed this warning may result in injury to the service provider, operator, or patient from electric shock, mechanical hazards, or other hazards.
-

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Maintenance Schedule

Perform this maintenance procedure **annually** to make sure the two heater reflector screws (front and back) are tight. This schedule recommends the minimum frequencies. Always follow hospital and local regulations.

WARNING: Failure to perform this maintenance per recommended frequency can result in patient harm or injury.

Required Tools

- One long Phillips screw driver
- One 5/16" wrench or nut driver
- One small step stool
- One 5/64" hex wrench
- One 3/8" nut driver

Heater Reflector Screws Maintenance Check Procedure

1. Turn off the Infant Warmer System and unplug its power cord from the outlet. Allow the heat tube to cool down before proceeding with the next steps.

WARNING: Disconnect the power to the warmer and allow the heat tube to cool to avoid the possibility of a burn.

2. Locate the warmer unit serial number on the back of controller.
3. For units that have serial numbers starting with the prefix HAK, AAN, HAJ, or HCA (see Figure 1):
 - a. Use a 5/64" hex wrench to remove the four mounting screws for the front cover plate, and remove the plate.
 - b. Use a 3/8" nut driver to remove the four mounting screws for the front heater housing cap, and remove the cap.
 - c. Proceed to step 5.

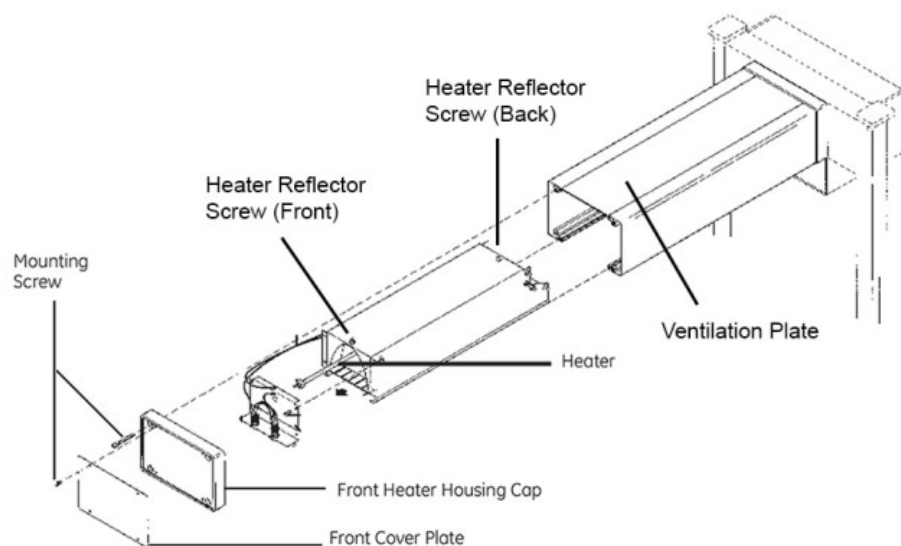
Figure 1: Heater Head Assembly – Units with HAK, AAN, HAJ, or HCA Serial Numbers

4. For units that have serial numbers starting with the prefix HCC (see Figure 2):
 - a. Use a Phillips screw driver to remove the screw and lock washer from the top of the cushioned heater head.
 - b. Use a Phillips screw driver to remove the screw and alarm light lens from the bottom of the heater head. Now the cushioned heater head can be pulled off the front of the unit.
 - c. Use a wrench to remove the four screws from the support plate end cap, and remove the plate.
 - d. Proceed to step 5.

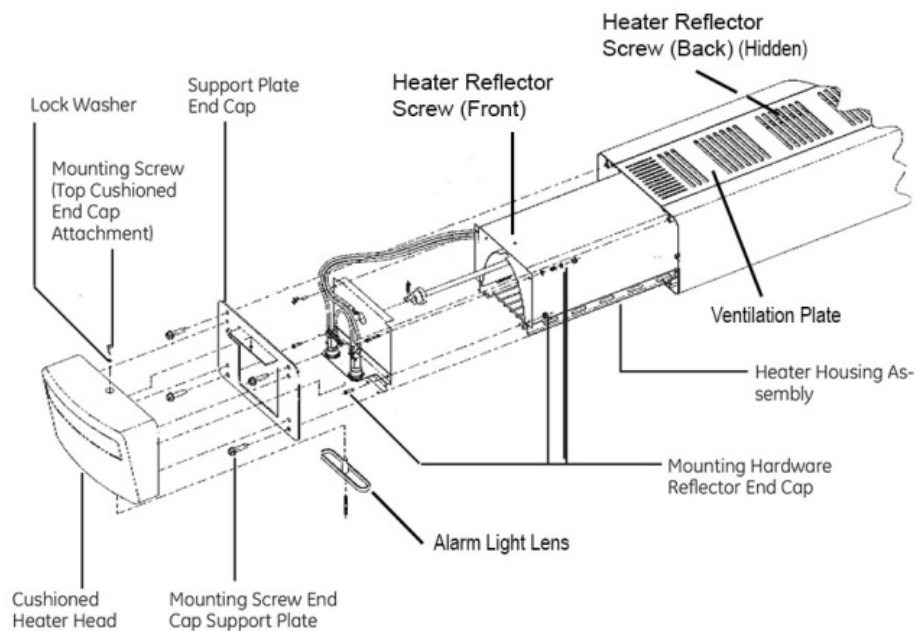
Figure 2: Heater Head Assembly – Units with HCC Serial Numbers

5. Pull out the ventilation plate on the top of the heater housing assembly (Figure 3).

Figure 3: Removing the Ventilation Plate

6. Locate the two reflector screw nuts (front and back) on the top of the heater housing assembly. (See Figure 4 and Figure 5).

Figure 4: Locating the Front Reflector Screw Nut

Figure 5: Locating the Back Reflector Screw Nut

7. Pass the tip of a long Phillips screw driver through the heater grille at an angle to engage the **front** reflector screw, and then use a wrench or nut driver to loosen the nut about one turn (Figure 6). Use care not to remove from screw. (You might need a small step stool to reach to the top of the heater assembly.)

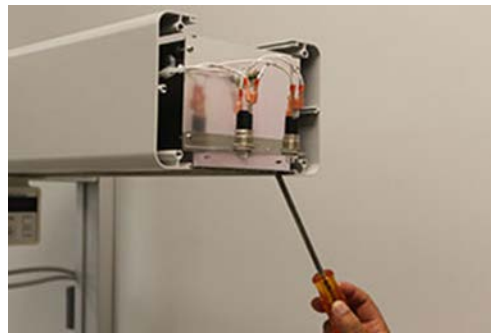
Figure 6: Engaging the Front Reflector Screw

8. While keeping the screw engaged with the screw driver, use your fingers to tighten the nut until it is snug.
9. Use a 5/16" wrench or nut driver to tighten the nut about an additional one-third of a turn (Figure 7).

Figure 7: Tightening the Nut in the Front Reflector Screw

10. Pass the tip of a long Phillips screw driver through the heater grille at an angle to engage the **back** reflector screw, and then use a wrench or nut driver to loosen the nut about one turn (Figure 8). Use care not to remove from screw. (You might need a small step stool to reach to the top of the heater assembly.)

Figure 8: Engaging the Back Reflector Screw

11. While keeping the screw engaged with the screw driver, use your fingers to tighten the nut until it is snug.
12. Use a 5/16" wrench or nut driver to tighten the nut about an additional one-third of a turn.
13. Re-install the ventilation plate into the system. Push the side tabs down on the ventilation plate, if necessary, for proper installation (Figure 9).

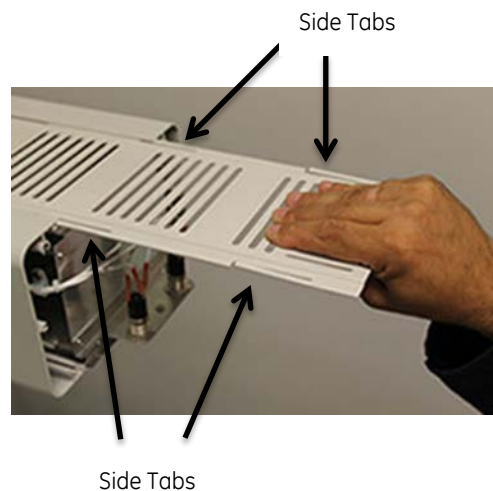
Figure 9: Re-installing the Ventilation Plate

14. Reverse step 3 or step 2 to re-install the heater head plate and cover.

NOTE: If the heater reflector screw or nut is lost, you can order the reflector screw by GE part number 0140-6124-114 and the nut by GE part number 6600-0382-400.

World Headquarters

GE Healthcare
9900 West Innovation Drive
Wauwatosa, WI 53226-4856
USA
Tel 1 800 345 2700

Europe, Middle East, Africa

GE Healthcare
P.O. Box 900
FIN-00031 GE
Finland
Tel +358 10 39411
Fax +358 9 146 3310

Latin America Representatives

GE Healthcare
3350 SW 148 Avenue
Suite 301
Miramar, Florida, 33027
USA
Tel + 1 954 744 5600

Brazil Only

GE Healthcare Clinical Systems Equipamentos Médicos Ltda
Av. Paulista, 37 - 13º andar
CEP: 01311-902 - Cerqueira César
São Paulo, SP - Brasil
Tel +55 11 3053 2500
Fax +55 11 3053 2573

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GE Medical Systems Türkiye Ltd. Şti
Esentepe Mah. Harman Sok. No: 8
34394 Sisli-Istanbul
Türkiye

 Ohmeda Medical,
a Division of Datex-Ohmeda, Inc.,
a General Electric Company
8880 Gorman Road
Laurel MD 20723
USA

EC Representative

GE Medical Systems SCS
283 Rue de la miniere
78530 BUC - FRANCE

Germany

GE Medical Systems Information Technologies GmbH
Munzinger Str. 3-5
79111 Freiburg
Tel. 49 761 4543 570
Fax 49 761 4543 571
Service 0800 4343258

Asia Representative

GE Healthcare
Shanghai GE (China) Hi-tech Park
No1 Huatuo Road, Zhangjiang Hi-tech Park Pudong, Shanghai,
P.R.China 201203
上海GE中国科技园
地址：中国上海市浦东张江高科技园华佗路1号, 201203
Tel + (8621) 38777888
Fax + (8621) 38777402

Australia	1300 722 229
China	800 810 8188
India	1 800 425 7255
Korea	(02) 1544 4564
South Eastern Asia	(65) 6277 3444

Addendum
Infant Warmer System Heater Reflector Screws
Maintenance Check
Printed in USA
© 2016 General Electric Company

2098189-001
Revision A