# Addendum

## Lullaby Warmer Preventive Maintenance Procedure for Heater Reflector Screws

For future reference, add this addendum to your Lullaby  
Warmer Service Manual (2042376-001)

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## Addendum - Preventive maintenance procedure for Heater Reflector screws

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## 1 Maintenance schedule

Perform this maintenance procedure **annually** to ensure the two heater reflector screws (north end and south end) are tight. This schedule recommends the minimum frequencies. Always follow hospital and local regulations.

**WARNING:-**

Failure to perform this maintenance per recommended frequency can result in patient harm or injury.

**Addendum - Preventive maintenance procedure for Heater Reflector screws**

**2 Introduction**

The main purpose of this document is to explain step by step preventive maintenance procedure for tightening the Heater Parabolic Reflector screws.

**Resource needed:**

- 1 Field Engineer /Bio med

**Tools needed:**

- One long Philips screw driver
- One M3 wrench or nut driver
- One small step stool
- Allen key Size 3 mm
- Allen key size 4mm, 1-Qty

**Equipment needed**

- No special equipment needed

In either case service personnel shall pay attention to below warnings.

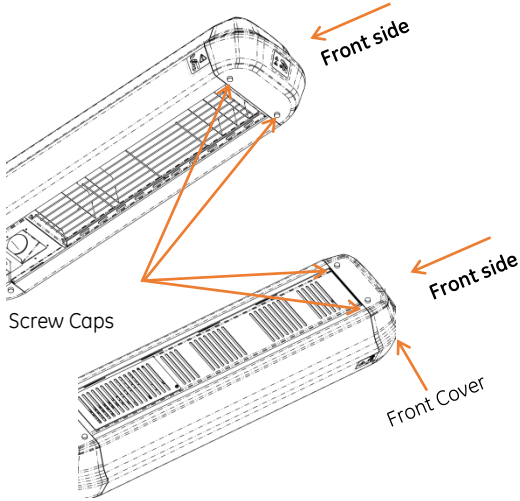
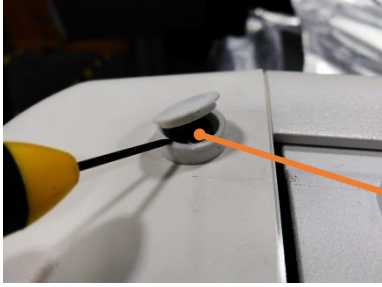
	<p><b>WARNING:-</b> Before replacing parts always do the following:</p> <ol style="list-style-type: none"> <li>1. Switch off the power.</li> <li>2. Set the castor brakes.</li> <li>3. Remove the cylinders if they are mounted on to the Lullaby Warmer.</li> <li>4. Always use anti-static material and wear anti-static wrist strap, safety glasses, safety shoes and safety gloves while dismantling the Lullaby Warmer to avoid possible damage due to static electricity and sharp edges.</li> </ol>
	<p><b>WARNING: -</b> Ensure power is OFF, before removing any covers for replacement of parts.</p>
	<p><b>WARNING:-</b> The heater assembly and the protective grid may still be hot if you dismantle the unit immediately after it has been in use.</p>
	<p><b>CAUTION:</b> Do not touch the inside of the heater reflector. Imprints of fingers may reduce the output of the heater’s reflective surface. The temperature of the heater can be determined by holding the skin temperature probe against the heater.</p>

### 3 Heater Reflector Screws Preventive Maintenance Check Procedure

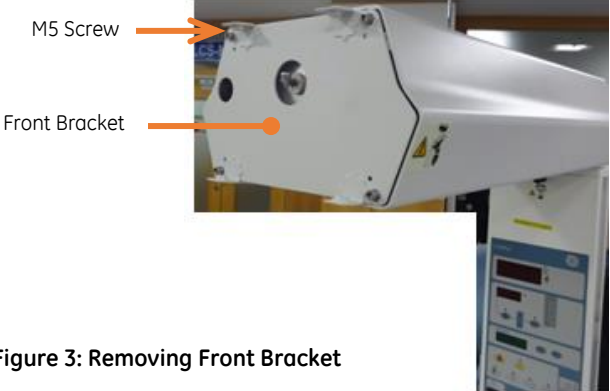
Follow the below step by step instruction to tighten the Reflector's M3 screws if screws are found loose.

Sl.No	Description	Illustration
	<p>Dis-assemble the Front cover, as follows;</p> <ol style="list-style-type: none"> <li>1. Open the <b>Screw Caps</b> at Front side of the <b>Hood assembly</b>. For screw cap locations refer Figure 2</li> <li>2. Use the small screw driver or any proper tool to open the Cap, as shown in the Figure 1.</li> </ol>	 <p style="text-align: center;"><b>Figure 2: Screw caps Location Front side</b></p>  <p style="text-align: center;"><b>Figure 1: Opening of Screw Caps</b></p>
	<ol style="list-style-type: none"> <li>3. Unscrew the <b>M5 screw (4-Qty)</b> to remove the Front cover.</li> <li>4. Keep the <b>Front Cover</b> at secure location.</li> </ol>	

Addendum - Preventive maintenance procedure for Heater Reflector screws

Sl.No	Description	Illustration
	5. Removing the front bracket: Unscrew <b>M5 screws (4-Qty)</b> ; remove the front bracket and the Teflon sheet (if provided) sheet.	 <p><b>Figure 3: Removing Front Bracket</b></p>
	6. Slide out the <b>Hood Top Plate</b> as shown in the Figure 4 . Keep it in secure location.	 <p><b>Figure 4: Hood Top Plate</b></p>
	7. Locate the Front (Figure 5) and Rear (Figure 6) M3 screws on the top of the Heater assembly.	 <p><b>Figure 5: M3 screw at Front side</b></p>  <p><b>Figure 6: M3 nut at Rear side</b></p>

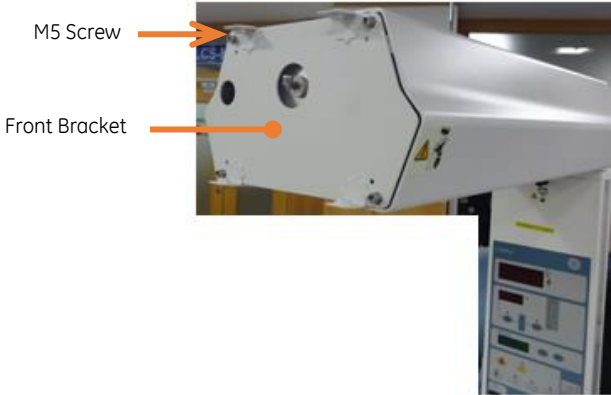
Addendum - Preventive maintenance procedure for Heater Reflector screws

Sl.No	Description	Illustration
	<p>9. Pass the Tip screw driver through the Grille.</p> <hr/>  <p><b>CAUTION:-</b> Carefully pass screw driver through the Grille. Ensure that no scratch and damage to the surface of Parabolic reflector</p> <hr/> <p>10. Firmly hold the Screw driver by pressing against the <b>M3 Screw</b>.</p> <p>11. While keeping the screw engaged with the screw driver, use your fingers to tighten the nut until it is snug.</p> <p>12. Use M3 nut driver to tighten the nut about an additional one-third of a turn.</p> <hr/>  <p><b>WARNING:-</b> Too much torque on the <b>M3 Nut</b> may cause damage to the reflector or <b>Heater assembly</b></p> <hr/> <p>13. Repeat the sequence from 7 to 11 for rear side screw, which is at rear side of Hood assembly. For details refer</p> <p>14. Figure 6.</p> <p>15. Ensure that both screws are tightened within the limits</p> <p>16. Slide-in the <b>Hood Top Plate</b>. Now you have successfully re-tightened the <b>M3 Screws</b>.</p> <hr/>  <p><b>WARNING:-</b> Ensure that there are not loose items left inside the Heater Assembly. Otherwise, it may cause hazardous situation to patient or equipment.</p> <hr/>	 <p><b>Figure 7: Tightening M3 screw at Front side of Hood assembly</b></p>

**Addendum - Preventive maintenance procedure for Heater Reflector screws**

Sl.No	Description	Illustration
	<p>17. Re-assemble the Front Bracket to the Hood assembly.</p> <p>18. Re-assemble the <b>Front bracket</b> to the front side of the <b>Hood</b>. Fasten the <b>Front bracket</b> with <b>M5 screws</b>. Tightening the screws till the washer bottom up. Figure 8.</p>	 <p style="text-align: center;"><b>Figure 8: Re-assembling Front Bracket</b></p>
	<p>19. Apply manual torque to re-assemble the <b>Front cover</b>.</p> <p>20. Close All 4 caps. You have successfully re-tightened the <b>M3 screws</b>.</p>	 <p style="text-align: center;"><b>Figure 9: Assembling Front Cover</b></p>

**Note: If reflector screws are missing, order heater assembly FRU**

- Part no 2043938-001 for 230 V with halogen lamp and transformer
- Part no 2062274-001 for 230 v with LED observation lamp and SMPS
- Part no 2056782-001 for 115 v with halogen lamp and transformer
- Part no 2061250-001 for 230V with LED observation lamp and SMPS

Replace the heater assembly with the New heater assembly as per the service manual (2042376-001) instructions provided in Chapter 7.

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© 2016 General Electric Company